

Prime TC Transport Chair Designed to be Easier to Clean

Situation

Hospital floors become contaminated with microorganisms from settling airborne bacteria by contact with shoes, wheels and other objects. The removal of such microbes is a component in controlling the healthcare-associated infections that can cost up to €7 billion per year in Europe.¹ Furthermore, in an investigation of the cleaning of hospital floors, the CDC found that just a few hours after floor disinfection, the bacterial count was nearly back to the pre-treatment level.² The wheelchair is one of the only pieces of equipment within a hospital that is utilised by nearly every department and nearly every patient. With the widespread use and important role that seated transport plays in the hospital setting, many are surprised to learn that the wheelchair was not designed for the hospital environment. Being the industry leader in patient mobility, Stryker Medical, in collaboration with the Michael Graves Design Group, redesigned this piece of patient transport equipment with microbial contamination in mind.

Technology

The Stryker Prime TC Transport Chair has several features that help to enhance a hospital's cleaning and disinfection procedures. Prime TC is made with a molded plastic, seamless design and powder-coated steel that eliminates fabric rips and reduces the risk of rust that can be difficult to clean. Foot operated flip-up footrests and a foot operated braking system allow for reduced bending and help prevent the transfer of germs and debris from touching dirty footrests and brakes. Prime TC is also power-washable which helps to make cleaning quick and easy.

Methodology

Lexamed, a third party biomedical testing laboratory, compared and assessed the buildup of soil on the Stryker Prime TC Transport Chair vs. a traditional wheelchair over the course of 25 cleaning cycles.³ The chairs were contaminated with a riboflavin solution (representing a simulated soil) and cleaned using a standard healthcare facility procedure utilising CaviWipes. The units were evaluated for residual riboflavin using fluorescence testing, and photographed throughout the 25 cycles.

Results

Overall, the Prime TC Transport Chair proved to be easier and more effectively cleaned than a traditional wheelchair typically found in a hospital setting. Specifically, the armrest, push handles, and seat were easier to clean on the Prime TC Transport Chair as compared to a standard wheelchair design. These areas showed little-to-no riboflavin build-up over time on the Prime TC Transport Chair.




Armrest After Cycle 25




Prime TC Transport Chair Traditional Wheelchair

Seat After Cycle 25

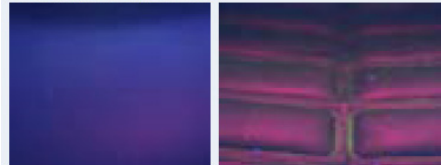

Prime TC Transport Chair Traditional Wheelchair

Push Handle After Cycle 25

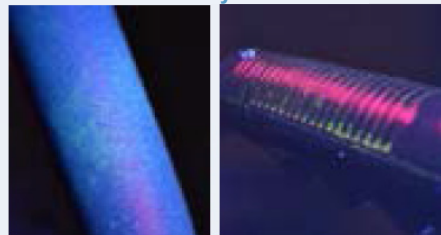

Prime TC Transport Chair Traditional Wheelchair

Residual riboflavin is shown as a bright blue/green against Prime TC Transport chair, and a bright green against the wheelchair.

Conclusion

Properly and effectively cleaning the seated transport device, a device that is frequently used throughout the hospital, is an important part of a hospital's cleaning and disinfection procedures. Lexamed concluded that the Stryker Prime TC Transport Chair is easier to clean than the traditional wheelchair.

References

1. (2014), World Health Organisation. Health care-associated infections. <http://ecdc.europa.eu/en/Pages/home.aspx> viewed April 20, 2015.
2. (2008). Guideline for Disinfection and Sterilization in Healthcare Facilities: Centers for Disease Control and Prevention. http://www.cdc.gov/hicpac/pdf/guidelines/Disinfection_Nov_2008.pdf viewed April 20, 2015.
3. (2014) Prime TC Cleaning Cycling Assessment Using Fluorescence Testing by Lexamed – Quality – Science – Innovation.

Reconstructive

Hips
Knees
Trauma & Extremities
Foot & Ankle
Joint Preservation
Orthobiologics & Biosurgery

MedSurg

Accessories
Computer Assisted Surgery
Endoscopic Surgical Solutions
Integrated Communications
Beds, Stretchers & EMS
Reprocessing & Remanufacturing

Neurotechnology & Spine

Craniomaxillofacial
Interventional Spine
Neurosurgical, Spine & ENT
Neurovascular

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Prime TC. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.



PRTCCS03EN
RRD/GS 2015/11

Copyright © 2015 Stryker

www.stryker.com



Stryker Medical
3800 E. Centre Avenue
Portage, Michigan 49002
USA

Distributed by:

Stryker UK
Hambridge Road
Newbury RG14 5AW
UK